UNIVET

TECHNE

Quick Guide

Thank you *for choosing us*

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Keep the original packaging;
in the case of a return or
maintenance, it will be necessary
to send the product to your
authorised dealer.

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Techné

In this “Quick Guide” you will find all of the useful information or the correct use and maintenance of your loupes. Additional component and accessory information can be found in the complete user guide. Please click the link below to access this information.

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Click this link to access information in languages not found in this guide.

Designed by Univet *in Italy*

The shapes and high-end materials that we use are defined by the Italian manufacturing tradition and they enrich all of our products, so that they become not only a tool, but a testimony of Italian craftsmanship excellence.

Since 1997 Univet has become an ambassador of style, quality and luxury, typical of the Italian fashion and design world. In the years we have created iconic shapes through which we distinguished ourselves, while we reconceived the idea of professional eyewear.

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IN-HOUSE *Design*

The project and the design are made in-house, they are a guarantee of unique and unmistakable aesthetics, innovation and comfort. These are imperatives features that have always characterized our products. The Univet loupes will become one with yourself to give you performance and comfort like never before.

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Techne

For years, Techne has embodied excellence in magnification systems – a timeless icon for professionals who demand more than clarity: they seek focus, comfort, and control. Today, Techne returns with refined ergonomics and elevated performance, designed to empower the way you work.

Techne is not a revival. It's a redefinition. Because true icons don't age. They evolve.

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In a magnifying system, the weight distribution it's one of the most important specs. Univet designs and creates frames, with an evenly distributed weight, that wraps in the most efficient way around the user's head, this means enhanced comfort even during prolonged use.

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Optics

Ottiche / Optik / Óptica / Optique

Univet optical elements are at the heart of the Loupes System and are the secret ingredient of its high performance. The optical elements are water and scratch resistant, and are equipped with an anti-glare treatment that guarantees the highest quality.

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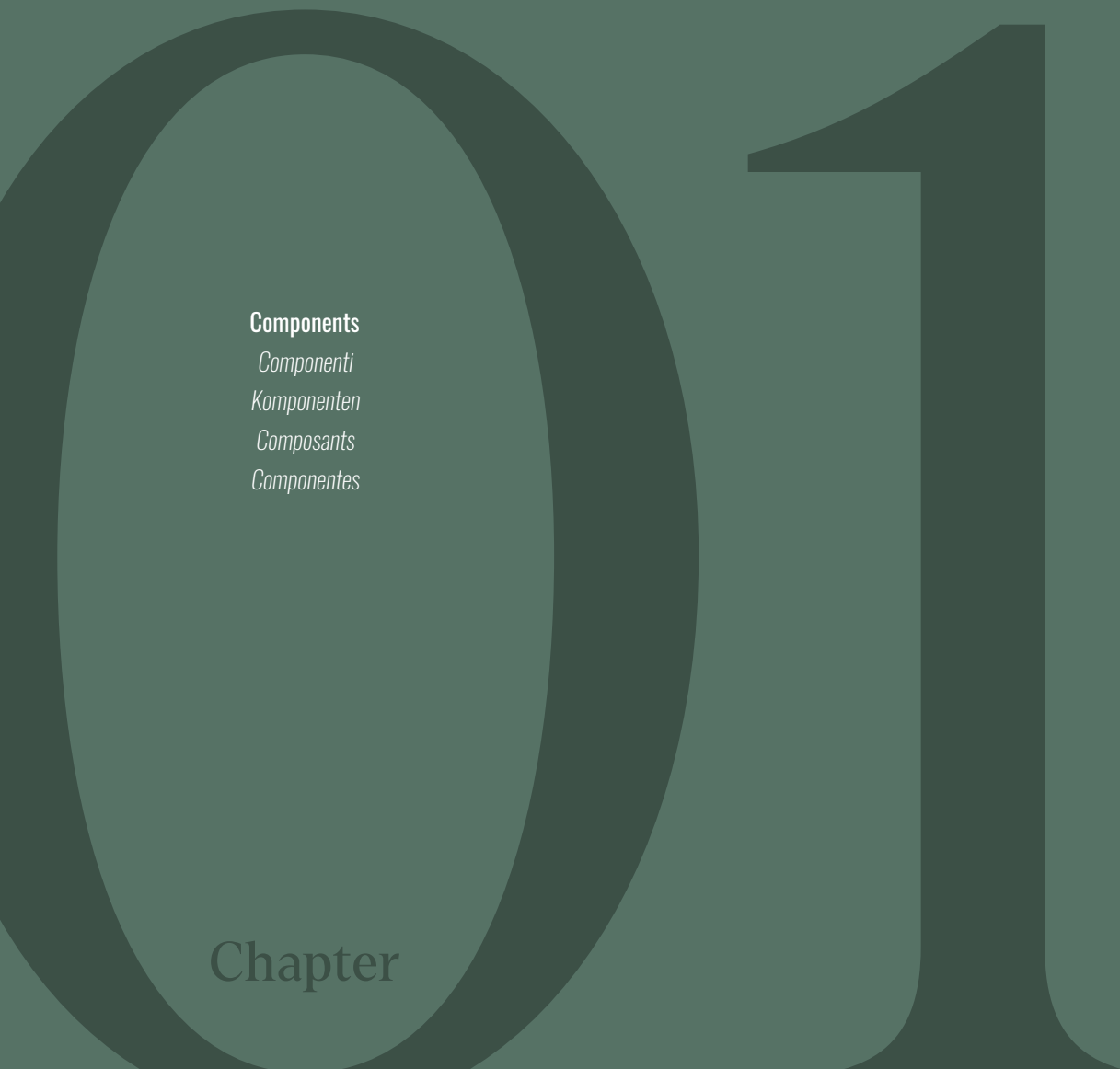
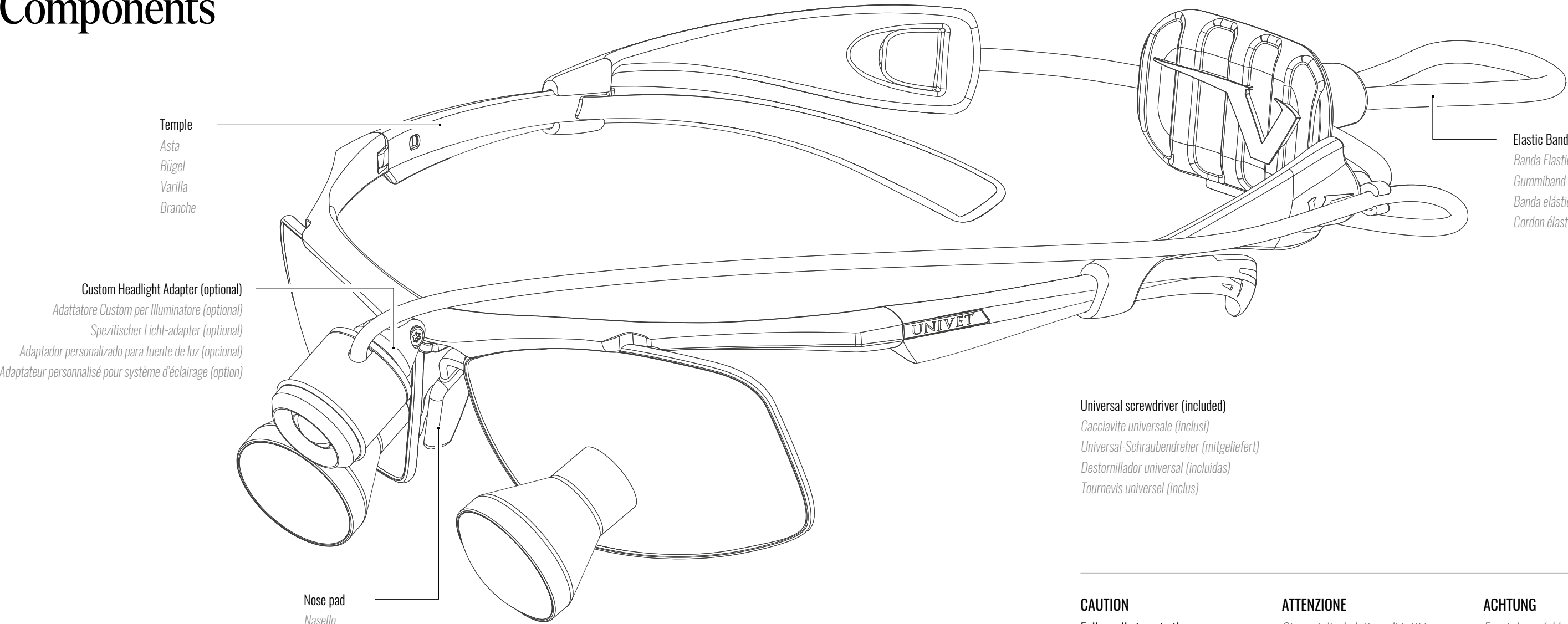
Tournevis universel (inclus)

CAUTION

Follow all steps in the sequence indicated for the optimum correct result.

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Correct Maintenance

Correct Maintenance

To maintain the optimum performance of your loupes over time, we suggest that you always store them inside the case provided and clean them regularly using the special spray liquid and cloth included in the package.

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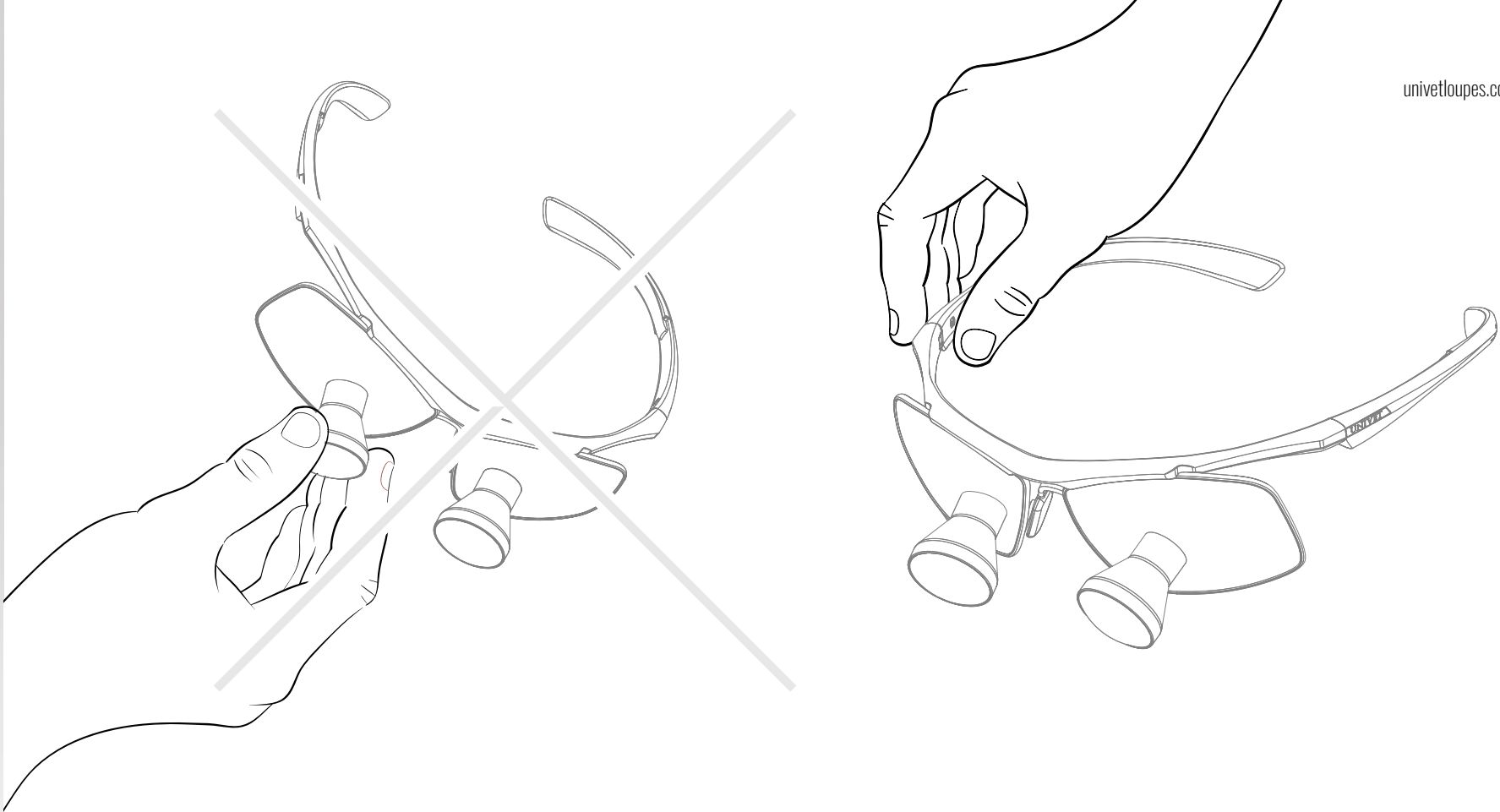
How to handle

How to handle

Important! Always handle the loupes by the frame, never by the binoculars; so that the optics are not compromised.

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Correct Vision

Correct Vision

In order to allow correct positioning through the optics, it is advisable to check that the position of the glasses are at the correct height.

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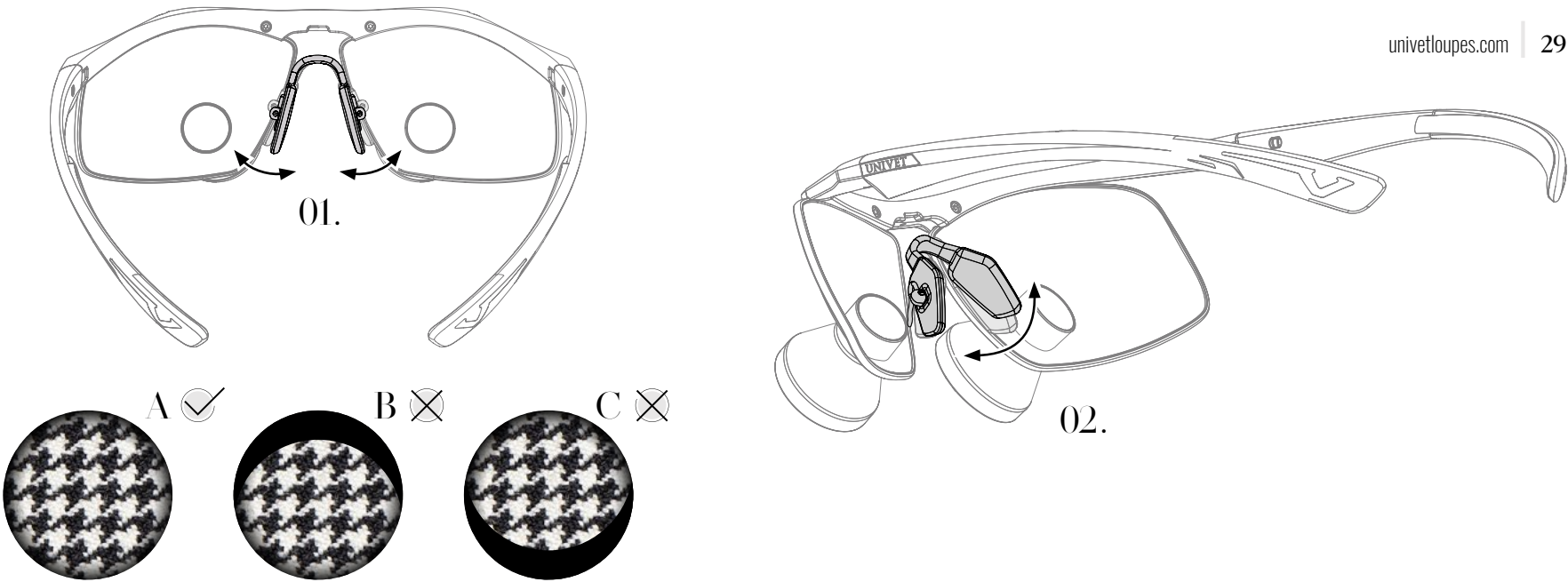
Height Adjustment

Height adjustment

It may be necessary to adjust the nosepiece to get the correct view "A". If the vision appears as in figure "B", it is necessary to tighten both sides of the nose piece. If the vision appears as in figure "C", it is necessary to "flood" them (figure 1). It is also possible to adjust the inclination of one or both sides of the nose pad (figure 02) to obtain the optimal fit.

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Correct Maintenance

More comfort

It is recommended to utilize the elastic band included in the package for maximum comfort. Particularly indicated in case of prolonged use.

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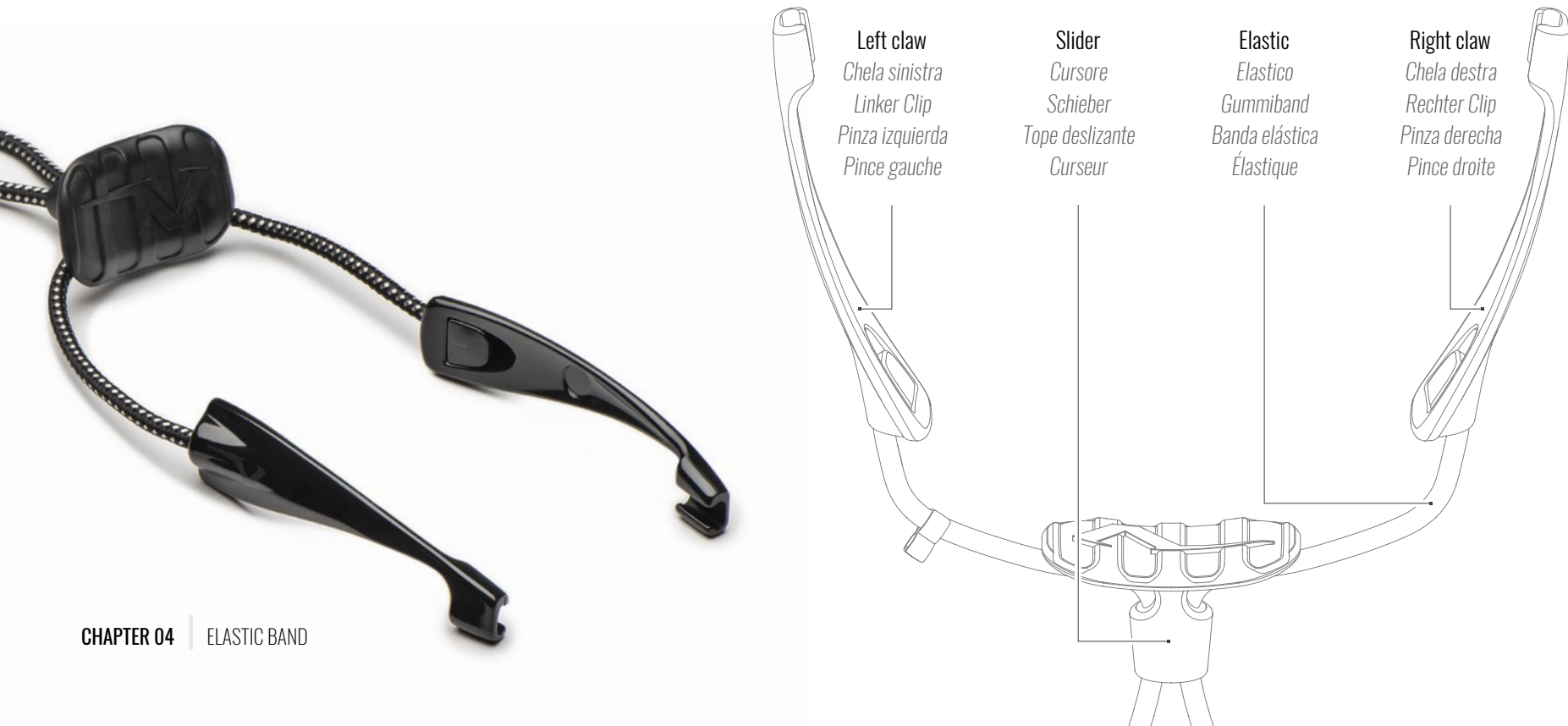
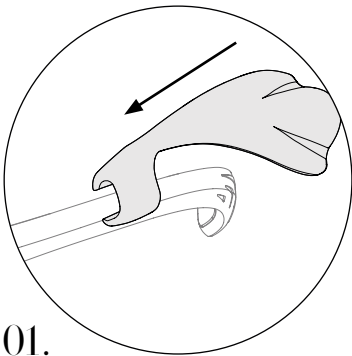
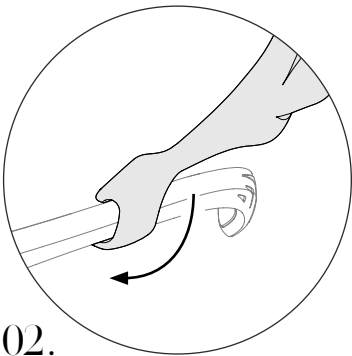
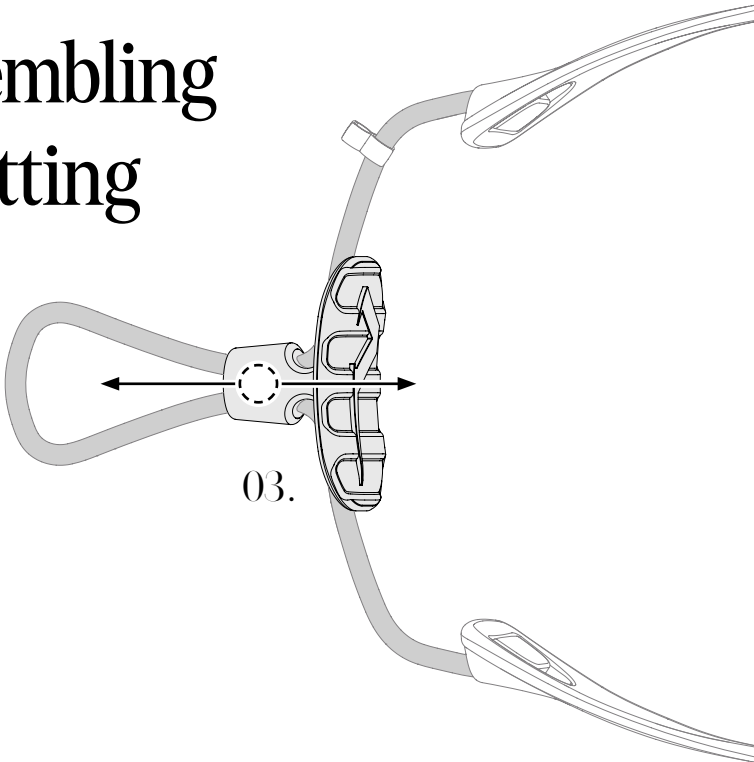
03.

Assembling & setting

To mount the elastic band on the glasses, hook the claw to the rod near the rubber end (Step 01) and then rotate it downwards to lock it (Step 02). Repeat the procedure on the other temple. To adjust the size of the elastic band, grasp the slider with two fingers and slide forward to tighten or backwards to loosen (Step 03).

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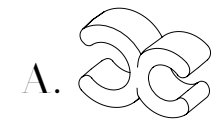
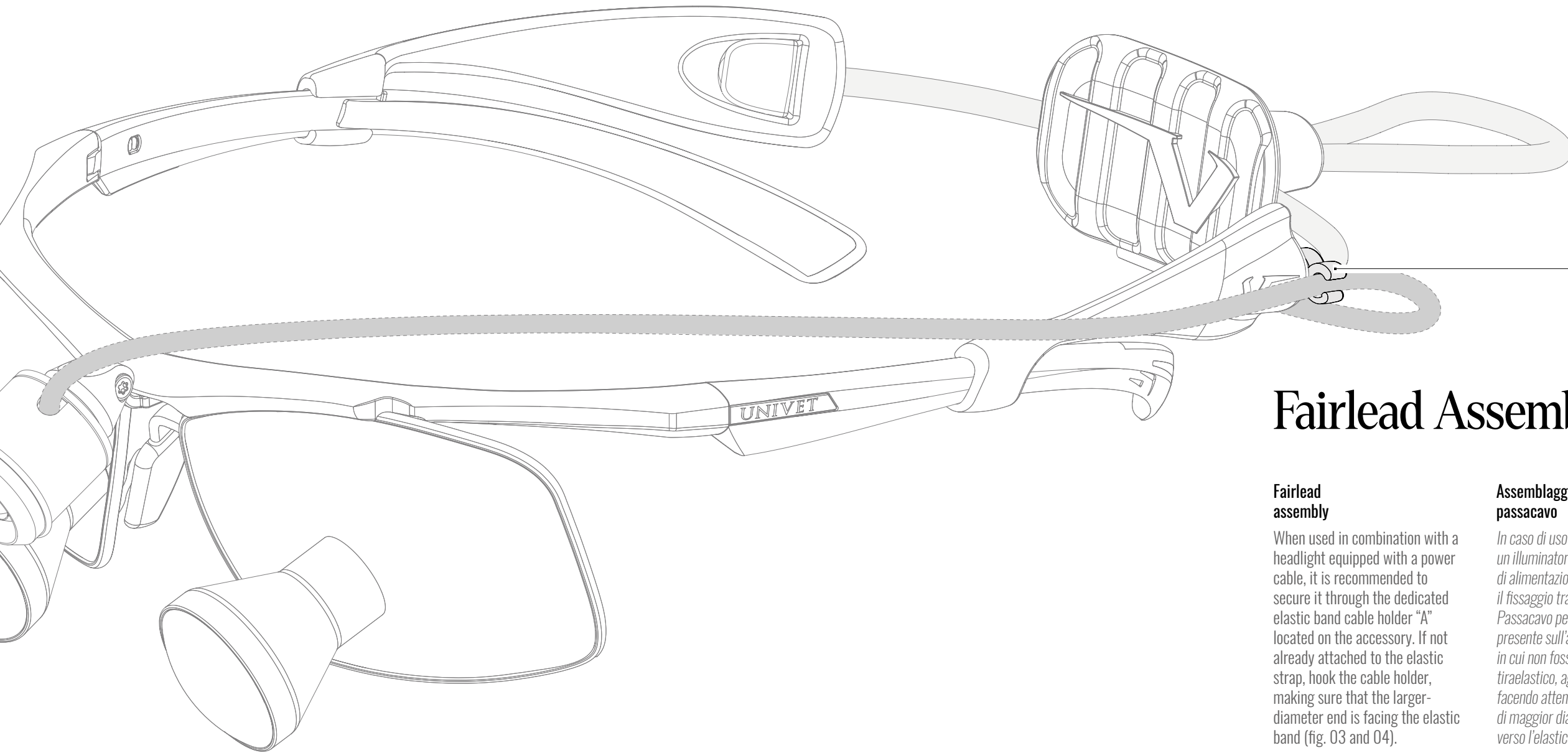
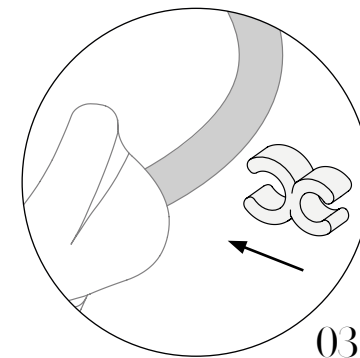
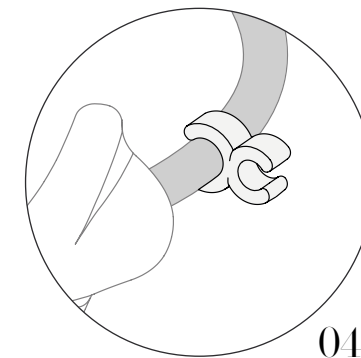
Fairlead Assembly

Fairlead assembly

When used in combination with a headlight equipped with a power cable, it is recommended to secure it through the dedicated elastic band cable holder “A” located on the accessory. If not already attached to the elastic strap, hook the cable holder, making sure that the larger-diameter end is facing the elastic band (fig. 03 and 04).

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All specifications and descriptions are subject to change without notice. All rights reserved. Text, images, graphics are subject to copyright and other protective laws. The content of this publication may not be copied, distributed or modified for commercial purposes without written authorization by Univet Loupes Spa.

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